## TMD Relief

2015-2016-2017  
Patented solution (EU)

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**TMD Relief**  
2015-2016-2017  
Patented solution (EU)

Instrument for exercising of the muscles of the mouth. Non Sterile Class I Medical Device

## WARNINGS, INDICATIONS, HOW TO USE

The device comprehends small part. If not properly secured to the device the small parts, may separate from the main body! Ingestion of this small parts may cause in breathing obstruction, be sure that all the moving parts are properly fixed to the device.

The device is for personal use only.

In some rare cases has been reported an increase in pain after or during the use of the device. If the exercise results in an increase of pain, stop the therapy and report to your doctor.

Do not change the device setting by yourself.

Follow your doctor's instructions about the therapy and the use of the device.

Before the use clean and sanitize the device, using a suitable sanitizer.

Periodically clean and sanitize the device, frequently disassemble and sanitize the blades.

Blades and grips can be sanitized in boiling water or using a suitable sanitizer.

Do not wash in a dishwasher or sanitize in boiling water the device plastic body!

To be used only under medical supervision and control.

## INDICATIONS


TMD Relief is suited to treat:

- Secondary tinnitus
- Bruxism
- Secondary tensive dental headaches
- Non otologic otalgia
- Atypical facial neuralgia
- Mandibular lock
- Pre-post intervention therapy
- Post operative rehabilitation




## CONTRA-INDICATIONS

TMD Relief is not to be used in presence of even suspicious skull bones problems, in particular regarding the maxilla and mandible, such as, but not limited to:


- Fracture of the maxilla and mandible
- Infections
- Osteonecrosis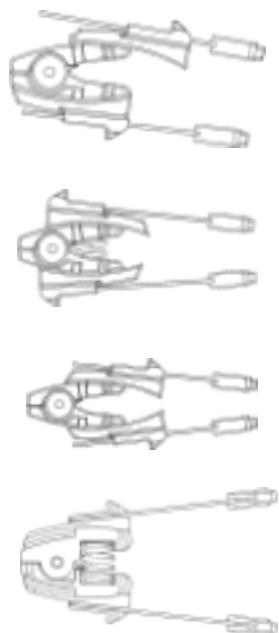

## DESCRIPTION OF THE DEVICE

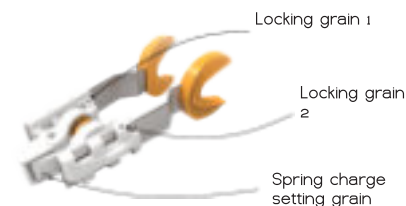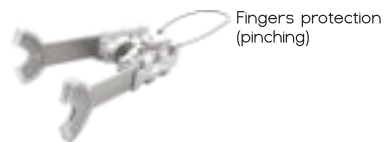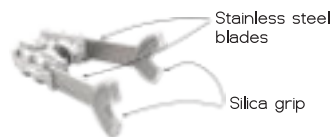Set the correct "non forced width" of the device by adjusting the "swings" according with the patients morphology.

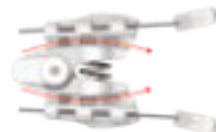Set the desired force/reaction of the exercise by adjusting the "blades".

For a stronger/softer reaction set the spring "charge setting grain".

## USE OF THE DEVICE

1. Remove from the package all the device parts.
2. Sanitize by boiling or with a suitable sanitizer, the blades.
3. Set strength and width.
4. Apply the security strap.
5. Insert the blades in your mouth so to let the teeth rest comfortably on the silica grips.
6. Exercise by closing your mouth then relaxing the facial muscles to allow the device to extend back to the "non forced width", corresponding to the "non forced opening" of the patient's mouth (dynamic exercise)
7. Alternatively, exercise by keeping your mouth open at a specific width (static exercise).

## THERAPY

According to the statistics the best results are reached within one month with a weekly exercise (2 minutes 5 times per day) to be repeated 2-3 times during the year under medical supervision and control.

Every week/15 days a visit to the doctor studio to set the device and to check the improvements due to the therapy is recommended.

## ORDER

Visit the website or scan the QR Code.

<https://bit.ly/2JqVadz>

NON-STERILE

PERSONAL USE ONLY

Once the silicon is worn, write to:

tmdrelief@syde.technology

We will retire the inox blades, after a specific treatment the blades will be reused saving energy and materials.

THANK YOU IN ADVANCE!

## TMD Relief

Instrument for exercising of the muscles of the mouth. Non Sterile Class I Medical Device

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